



HD EEZI-TRANSFER™ – CURVED SITTING BOARD XL

This guide covers products with the code HDCRVXL02.

The **Curved Sitting Board XL** is used for sitting transfers between surfaces that are ideally at a similar level, for example: from bed to wheelchair; wheelchair to commode; wheelchair to car seat etc. However, sometimes the giving surface (bed shown here) is slightly higher than the receiving surface (wheelchair shown here).

USER INSTRUCTIONS

SWL
450KG



6-4 FIM
ASSESSED



1

WARNING: Before using check that the distance between the two surfaces does not exceed 20cm gap and that the non-slip pads on the base of the board are secure on both surfaces.

Slide one end of the **Curved Sitting Board XL** under the buttocks (the smooth tapered edges of the board helps here). For some people it may help to lean slightly away from the board so the buttock is raised slightly and the board can be inserted more easily. Check that the other end of the board is positioned securely on the receiving surface i.e. in this example the wheelchair.

2



3

The user can now slide easily across the **Curved Sitting Board XL** from one surface to another i.e. the bed to the wheelchair. The curve of the board helps the person's body to turn so that they can sit comfortably on the receiving surface.

Please note: the user must have upper sitting balance to be able to carry this task out independently, otherwise they will need assistance from one or two carers, depending on need.

Once on the receiving surface the user can slide the **Curved Sitting Board XL** out easily. This can be helped by leaning slightly away from the board to raise the buttocks a little and remove the board.

4



PRODUCT FEATURES

- All edges tapered for user comfort and ease of insertion and removal.
- Two robust anti-slip strips grip any surface and provide extra security and safety for the user.
- Comfortable carry handle.
- Superb Safe Working Load (SWL) of 450kg.



MATERIAL CONTENT

100% HDPE


STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.



WARNINGS AND PRECAUTIONS

- Do not leave on the floor when not being used.
- Inspect before use, including the non-slip panels, for signs of wear or damage
- Always test product grip on your chosen surfaces (bed, wheelchairs, car seats etc.) before using.
- Safe Working Load (SWL): 450kg.
- The gap should not exceed 20cm.

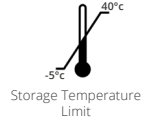

CLEANING INSTRUCTIONS

- Wipeclean only.
- Wipeclean with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.



DISPOSAL INSTRUCTIONS

- Fully recyclable.
- If it has been used by an infectious person, thoroughly disinfect before recycling.
- Otherwise dispose of through normal recycling waste management.



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, B-1030 Brussels, Belgium



www.hospitaldirect.co.uk


HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP

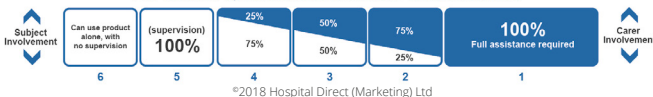

EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.