



GUS™ SLIDE – DISPOSABLE / SINGLE PATIENT USE

This guide covers all products with the prefix DGUS in the product code

The **GUS™ Slide – Disposable** is a Gynaecology / Urology, and Head-Down Slide is essential for safe positioning when performing Lithotomy and other head-down procedures.



USER INSTRUCTIONS

It is important, that the patient's back area is covering the anti-slip area (seen by a stitch line). This is not designed to cover the buttocks. To position correctly follow these instructions:



To position correctly, place at the break of the table, as shown, then slide up to the head end.



Make sure the yellow flaps are tucked in to keep the **GUS™ Slide** able to slide.



In this image the **GUS™ Slide** is in the correct starting position for the patient.



Add pillow and pull out the yellow panels on each side to lock the product.




The **GUS™ Slide** should look like this prior to patient.



Patient in position, product locked, prior to moving down the table.



Use **Slippy Sleeves** to tuck in yellow flaps.

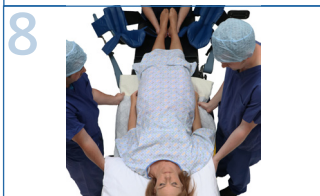

Slide patient further down the table. Once at the break, stop.



Position patient correctly and pull out yellow flaps to lock position. It is now safe for head-down positioning up to 25 degrees.

POST SURGERY: To move the patient back up the table: **(1)** Level out the table to a horizontal position. **(2)** Re-insert the slippy panels to make the product slide. **(3)** Remember to use the **Slippy Sleeves** – they make the process so much easier. **(4)** Once both sleeves are re-inserted, you can now slide the patient back up to the head of the table. **(5)** Pull out the slippy panels again to lock the patients position. **(6)** The patient can now be transported to Recovery on the **GUS™ Slide** and can remain on it until no longer necessary. If you wish to sit the patient up in Recovery, simply pull out the panels afterwards from each side to LOCK. **(7)** If at any point, when the **GUS™ Slide** is in slide mode (i.e. with the slippy panels not hanging down) and the patient needs to be turned – simply use the **GUS™ Slide** as a slide sheet to position the patient on their side.



SUPPORT TRAINING: On-site and Online. Please contact us for more information.

EACH PACK CONTAINS (10 PACKS PER BOX)

- GUST™ Slide – Disposable/Single Patient Use x 1
- GUST™ Slippy Sleeves – Disposable/Single Patient Use x 1 pair

MATERIAL CONTENT

- Quilting Top Layer: 100% Non-Woven Polyester
- Inner Layer: 100% Polyester Wadding
- Yellow Panels: 100% Polyester
- Anti-slip Layer: PVC coated polyester, phthalate free
- Slippy Mitts: 100% Polyester

STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Safe for head-down positioning up to 25 degrees.
- Do not leave or drop on the floor; it is a trip hazard.
- Inspect product, including handles, before use for signs of damage or tears.

CLEANING INSTRUCTIONS

- Wipeclean, if needed for the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Single Patient Use



Keep dry



Keep away from sunlight



Caution



Do not use if package is damaged



Storage Temperature Limit



LATEX



Patient Information website



Medical Device



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT


Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.

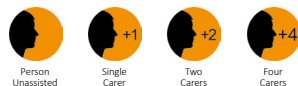

HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.