



EXPERTS IN PATIENT HANDLING SOLUTIONS



# GLIDE CUSHION / IN-BED SELF TURN

This guide covers all products with the prefix GC in the product code.

The **Glide Cushion / In-Bed Self Turn** is an innovative patient turning system, helping patients to rotate themselves independently.

## USER INSTRUCTIONS



# 1

Unfold the **Glide Cushion** and place on the bed, towards the top end of the bed near to the pillows.

Make sure the soft quilted side is on the outside and the slippery finish is on the inside.

It is now easy for you to turn yourself in bed with ease into a new position and sleep on the **Glide Cushion** throughout the night.  
ONLY SUITABLE FOR PEOPLE WITH NO RISK OF FALLING OFF THE BED.

# 2

## USER INSTRUCTIONS (Protective Wipeclean Cover)

# 1

Place the super-soft specialist fabric cover around the **Glide Cushion**. This extra layer of softness will also protect against spills/leaks.

It is now safe and comfortable to sleep on throughout the night.

# 2

# 3

It is now easy for you to turn yourself in bed with ease into a new position.

## MATERIAL CONTENT

Outer fabric: 100% Polycotton  
 Filling: 100% Polyester  
 Inner Fabric: 100% Polyester  
 Cover: Breathable Coated Nylon



Glide Cushion Cover

## STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.



Keep dry



Keep away from sunlight

## WARNINGS AND PRECAUTIONS


- Do not leave or drop on the floor; it is a slip hazard.
- Test for slippiness before use. If over-laundered and doesn't slide please discard or replace.
- Inspect product before use for signs of damage or tears..



Caution



Do not use if package is damaged

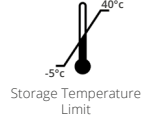

Storage Temperature Limit  
 40°C  
 -5°C

## CLEANING INSTRUCTIONS

- Wash as per laundry instructions.
- Can be wiped clean, between use with the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable



Wash at 30-74°C. Tumble dry on low heat. Do not Iron.  
 Do not use bleach. Do not use fabric conditioner.

## DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Patient Information website



Medical Device



### MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



[www.hospitaldirect.co.uk](http://www.hospitaldirect.co.uk)


## HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

### FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

#### FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP

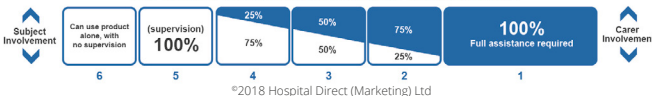

### EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.