



EXPERTS IN PATIENT HANDLING SOLUTIONS



HD DELUXE™

HD DELUXE™ LIMB PAD – FOLDING POSITIONING WEDGE

This guide covers the product code with the product code LIM40.

Our **HD Deluxe™ Limb Pad – Folding Positioning Wedge** can help provide pressure relief and extra comfort in many different lying positions. At only 5cm thick it can be positioned under, shoulders, legs, ankles and even folded to give extra thickness where required. Plus, it can be combined with other limb pads from our range depending in your needs, Below are some examples of how it can be used.


USER INSTRUCTIONS



Place the **Limb Pad – Folding** under the elbow and forearm. This will provide fantastic support and additional comfort when lying on your back as it reduces the pressure points throughout the arm and up to the shoulder.



The **Limb Pad – Folding** can offer excellent comfort when positioned under the heels as it allows the heels and ankles to rest into the pad. This can help with ankle and skin care rehabilitation.

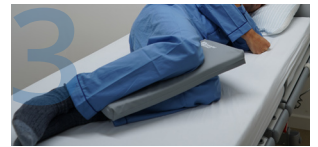

Rest the **Limb Pad – Folding** in between both legs so the knees and shins are supported. This will reduce the pressure points between both knees and encourage a better sleeping position when lying on your side.



Fold the **Limb Pad – Folding** in half and rest it in between both legs in line with the knee joints. It will stop the knees rubbing together and provide additional comfort when lying supine.

The **Limb Pad – Folding** can also be used in between the knees without folding.



Use the **Limb Pad – Folding** on top of the **Limb Pad – Rectangle** if you require slightly more height to elevate the legs or require extra support.



For maximum patient comfort and support add the thinner **Limb Pad – Folding** under the elbow joint to reduce the pressure points down the shoulder and forearm. Two **Limb Pad – Angular** can also be used with the **Limb Pad – Rectangle** to elevate the legs and support the ankles.

FABRIC CONTENT

Cover Fabric: 100% polyester
Cover Coating: 100% Polyurethane Coating
Foam: Polyurethane Foam



STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Do not leave the on the floor when not being used.
- Do not use on a wet floor.
- Always test product grip on your chosen surface before using.
- Inspect before use for signs of wear or damage.



Keep dry




Keep away from sunlight



Caution



Do not use if package is damaged

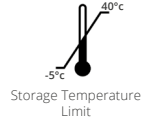

CLEANING INSTRUCTIONS

- Wipeclean with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Wipeclean Only



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT


Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.

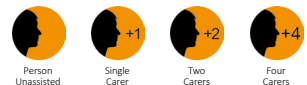

HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.