



EXPERTS IN PATIENT HANDLING SOLUTIONS




HD DELUXE™

HD DELUXE™ LIMB PAD – RECTANGLE POSITIONING WEDGE

This guide covers the product code with the product code LIM150.

Our **HD Deluxe™ Limb Pad – Rectangle Positioning Wedge** can help provide pressure relief and extra comfort in many different lying positions. It can also help a carer manage personal care and bandaging more easily. It can be used on its own or in combination with other limb pads from our range depending on your needs. Below are some examples of how it can be used.

USER INSTRUCTIONS

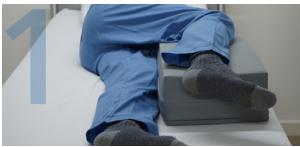

With the patient in a side lying position place the **Limb Pad – Rectangle** under the knee and shin. This will provide the carer with greater access for personal care and make the patient more comfortable in a side lying position.



Now that the leg is resting on the **Limb Pad – Rectangle** an additional Wedge can be used under the arm of the patient providing additional comfort and security for the patient, especially for those with dementia. The extra limb pad shown is our **Limb Pad – Angular**.



Place the **Limb Pad – Rectangle** under the patient's calf making sure that it is close to the ankles. This can help offload any pressure relating to a sprain or an injury, but also for personal hygiene and sock / dressing bandage application.



The **Limb Pad – Rectangle** and the **Limb Pad – Angular** are the same height so they can be used together. Place the **Limb Pad – Angular** under the thighs and the **Limb Pad – Rectangle** under the patient's calves so the ankles are raised. The head section of the bed can then be profiled. Together they will form a knee-break and stop the patient moving down the bed.



For maximum patient comfort and support add the thinner **Limb Pad – Folding** under the elbow joint to reduce the pressure points down the shoulder and forearm. Two **Limb Pad – Angular** can also be used with the **Limb Pad – Rectangle** to elevate the legs and support the ankles.



If the **Limb Pad – Rectangle** isn't quite high enough place the thinner **Limb Pad – Folding** on top of the larger one so it increases the height elevation for the patient to make them more comfortable.

MATERIAL CONTENT

Cover Fabric: 100% polyester
Cover Coating: 100% Polyurethane Coating
Foam: Polyurethane Foam

STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Do not leave the on the floor when not being used.
- Do not use on a wet floor.
- Always test product grip on your chosen surface before using.
- Inspect before use for signs of wear or damage.

CLEANING INSTRUCTIONS

- Wipeclean with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Keep dry



Keep away from sunlight



Storage Temperature Limit



Caution



Do not use if package is damaged



Wipeclean Only



Medical Device



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



www.hospitaldirect.co.uk


HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP

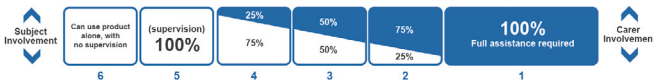

©2018 Hospital Direct (Marketing) Ltd

EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.