



# MORTUARY FRIDGE BAND

This guide covers all products with the prefix MFB in the product code.

The **Mortuary Fridge Band** is for use by mortuary technicians to secure arms and legs of deceased within the confines of their fridge tray.



## USER INSTRUCTIONS

	<p><b>1</b> Unfasten the black buckle if it's connected so the band lays flat. Turn the band over so the webbing and buckles are on the underside.</p>
<p>Slide the band under the body at either the head or foot end and then slide to the area that needs supporting.</p>	
	<p><b>3</b> Before connecting the band, make sure there's an equal length both sides.</p>
<p>Bring together both halves of the band round the deceased and fasten securely using one of the two buckles which best fits that particular body.</p>	
	<p><b>5</b> If the band is still loose further adjustment can be made by pulling the black webbing strap through the buckle until the band is securely fastened.</p>
<p>Once the band is finished with unfasten the buckle and slide the product out from under the body.</p>	

## MATERIAL CONTENT

**Fabric:** 100% Nylon

**Filling:** Polycotton/Polyester Wadding

**Handles:** Polypropylene

## STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

## WARNINGS AND PRECAUTIONS

- Do not leave the on the floor when not being used.
- Inspect the product, including handles, before use for signs of damage or tears.

## CLEANING INSTRUCTIONS

- Wash as per laundry instructions.

## DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Keep dry



Keep away from sunlight



Caution



Do not use if package is damaged



Storage Temperature Limit



Wash at 30-74°C. Tumble dry on low heat. Do not iron. Do not use bleach. Do not use fabric conditioner



Patient information website



Medical Device



### MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



[www.hospitaldirect.co.uk](http://www.hospitaldirect.co.uk)


## HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

### FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

#### FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP

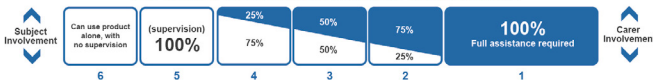

©2018 Hospital Direct (Marketing) Ltd

### EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



Person Unassisted



Single Carer



Two Carers



Four Carers

HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.