



EXPERTS IN PATIENT HANDLING SOLUTIONS



EASIDRESS – DISPOSABLE

This guide covers all products with the prefix PSDP.

The **EasiDress – Disposable** allows patients that are recovering from an injury or surgery and need some assistance, to get help to dressed.

USER INSTRUCTIONS



1

Slide the **EasiDress** arm sleeve over the arm you need to dress. Then slide the pyjama top, blouse or jumper sleeve on over the arm sleeve, as far up as you can.

Once you have the garment in place, remove the **EasiDress** arm sleeve carefully. Repeat with the second arm and then remove the remaining arm sleeve.

2



3

To dress the legs, bring the **EasiDress** leg sleeve right up as far as you can and gently slide pyjamas, trousers, jeans, or shorts over the **EasiDress** leg sleeve. Repeat if necessary with the other leg



From the foot end, gently pull and remove the **EasiDress** leg sleeve, and repeat for the remaining leg.

4



PACK INCLUDES

- 1 x EasiDress – Disposable Leg Sleeve
- 1 x EasiDress – Disposable Arm Sleeve

MATERIAL CONTENT

100% Polyester

STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS


- Do not leave or drop on the floor; it is a slip hazard.
- Inspect product, before use for signs of damage or tears.

CLEANING INSTRUCTIONS

- Wipeclean, if needed for the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.

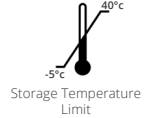

MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, B-1030 Brussels, Belgium



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.